

GET SET-UP IN 3-STEPS | DexcomG6



Did you receive a Hello Dexcom sample kit?

Before you start, activate your Hello Dexcom by scanning the QR code on the right or visit dexcom.com/hello.



You will receive a guided email journey with helpful tips on using your Dexcom G6, so you can make the most of your first 10 days!



Check your smart device[†] compatibility online at dexcom.com/compatibility.

Getting Started:

1

Download and open the  **Dexcom G6** app using your compatible smart device[†]

Scan for Android app



Scan for iOS app



2

A Follow on-screen setup instructions

Sensor code is found on the applicator

Transmitter Serial Number (SN) is found on the box or back of the transmitter



OR



B Insert your sensor and transmitter, then wait 2 hours for sensor warm-up



Follow the guided instructions on your Dexcom G6 application

OR

Watch the tutorial on YouTube

Dexcom Canada - How to insert the sensor and transmitter



If you need assistance with your G6 CGM, call **1-844-832-1810 (toll free)**

Dexcom CARE option 4

Mon-Fri 9am-5:30pm EST[†]

Receive personal CGM training and support from certified diabetes educators:

- Getting started on your Dexcom G6
- Sensor insertion steps
- Help using the G6, Follow and CLARITY app features

For complete instructions on how to set up and to use your Dexcom G6, please refer to your Start Here Guide and Using Your G6 Guide

Overview of your Dexcom G6 and key features:



Trend Arrows

Your trend arrows show you where and how quickly your glucose is changing in the next 30 minutes so you can take action ahead of time.

Alerts and Alarm

Your alerts and alarm are designed to bring your attention to your glucose levels for when action may be needed.

Change your alert settings from the Dexcom G6 app. Go to **Settings > Alerts**

For training videos, handy guides, FAQs and more, visit:
dexcom.com/en-ca/training-resources

3

Get the most from your Dexcom G6 by using the following apps



Dexcom CLARITY

Receive key glucose metrics, view and download PDF reports, authorize data sharing with your healthcare professional and more.

Scan for Android app



Scan for iOS app



Dexcom Follow[†] (Friends and Family app)

Up to 5 people can follow your glucose levels, giving you the support you need. Ask your loved ones to download the Dexcom Follow[†] app.

Scan for Android app



Scan for iOS app



Have more questions? Call **1-844-832-1810 (toll free)**

Customer Service **option 1**

Mon-Fri 9am-8pm EST[§]

Sales and phone order support, phone compatibility help

Insurance Support **option 2**

Mon-Fri 10am-8pm EST[§]

Our dedicated team can easily verify the insurance opportunities available to you

Technical Support **option 3**

24/7

Assistance with product questions and troubleshooting

* If your glucose alerts and readings from the Dexcom G6 do not match symptoms or expectations, use a blood glucose meter to make diabetes treatment decisions.

† Smart Device sold separately. For a list of compatible smart devices, please visit dexcom.com/compatibility.

‡ Separate Follow app and internet connection required. Users should always confirm readings on the Dexcom G6 app or receiver before making treatment decisions.

§ Hours are subject to change and exclude holidays.

Dexcom, Dexcom G6, Dexcom Follow, Dexcom Share, and Dexcom CLARITY are registered trademarks of Dexcom Inc. in the United States and may be registered in other countries.

© 2021 Dexcom Canada, Co. All rights reserved. LBL021747 Rev002

Dexcom