## Critical safety notices!

### Warning

#### Turn On Dexcom ONE+ optional alerts

- No glucose alerts unless you turn them on.
- Turn on optional alerts during setup to get glucose and system alerts when sensor warmup is complete.
- If you don't turn on optional alerts, you might miss a severe low or high glucose event.

#### Dexcom ONE+ alerts won't override your display device settings

- If your display device is on silent or vibrate, you won't hear high or low glucose alerts.
- Turn on display device sound to hear alerts.
- Check your display device sound settings, or you may miss an alert.

## Receiver safety information

### Warning

Not using supplied USB charger and cable may cause the receiver battery to not charge. Don't use if the supplied USB charger or cable is damaged. Store supplied USB charger and cable safely. Misuse of the USB cable can be a strangulation risk.

### Precaution

Test your receiver speaker and vibrations regularly. To make sure the speaker and vibrations work, plug in the receiver to charge. The Speaker Test screen appears for a few seconds. Follow the directions on the screen to test the speaker and vibrations. If it doesn't beep and vibrate, contact Dexcom Technical Support at [dexcom.com](http://dexcom.com) or contact your local Dexcom representative and use your app or BG meter until the receiver is fixed.

Keep receiver clean and dry: Don't submerge your receiver in water and don't get dirt or water in the USB port. That could damage it.

## Set up your Dexcom ONE+

To use your system safely, read all product instructions before using your Dexcom ONE+, including the Safety Statements in the sensor box and the **Dexcom ONE+ User Guide**, available online at [dexcom.com](http://dexcom.com).

### 1 Turn on receiver

The button with the circle on it is the **Select** button.

To turn on the receiver, press and hold the **Select** button for 3–5 seconds.


### 2 Set up your receiver

- Follow the instructions on receiver screens to set up your receiver.
- Insert your sensor using the Insert Sensor instructions in the sensor box.
- After inserting and pairing your sensor, follow local guidelines for throwing out the applicator and recycling the Dexcom packaging.

### 3 Sensor warmup

Your sensor has to warm up before you get sensor readings. The sensor warmup timer tells you when your sensor readings and alerts will begin.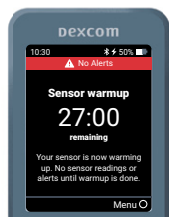

### 4 Your sensor session

- Don't get sunscreen or insect repellent on your receiver.
- Update the date and time on the receiver when needed.
- To learn more about using your Dexcom ONE+, go to [dexcom.com](http://dexcom.com), where you'll find the Dexcom ONE+ User Guide and more.

## Receiver navigation

Navigate using the four buttons at the bottom of the receiver.

- Press the **down arrow**  $\nabla$  to scroll down the screen.
- Press the **up arrow**  $\wedge$  to scroll up the screen.
- To scroll faster, press and hold an **arrow** button.
- To move to the next field, use the **Select** button.

## Receiver home screen

After warmup, the home screen shows your sensor readings and your glucose trend information.

- Number:** Your most recent sensor reading.
- Trend arrow:** Where your glucose is going based on the past few readings.
- Trend graph:** Big dot on the right is your most recent sensor reading. The smaller dots to the left are past readings.
- 3 Hours:** This is the number of hours shown on your graph. Change this using the up or down arrow buttons.
- Target range (shaded rectangle inside graph):** 3.9–10.0 mmol/L is the international consensus for recommended target range.
- High alert yellow line:** You get your High alert when your glucose is at or above this yellow line. Change level in **Menu > Settings > Alerts > High**
- Low alert red line:** You get your Low alert when your glucose is at or below the red line. Change level in **Menu > Settings > Alerts > Low**

For more information, go to **Menu > Help**. You can also find information, instructions, tutorials, and the Warranty at [dexcom.com](http://dexcom.com).

## Trend arrows

Trend arrows can help you predict where your glucose will be within the next 30 minutes.

- Steady:** Changing less than 1.7 mmol/L in 30 minutes
- ↗ Slowly rising or falling:** Changing 1.8–3.3 mmol/L in 30 minutes
- ↘**
- ↑ Rising or falling:** Changing 3.3–5.0 mmol/L in 30 minutes
- ↓**
- ↕ Rapidly rising or falling:** Changing more than 5.0 mmol/L in 30 minutes
- ↕↕**
- ↕↕↕ No arrow:** Can't determine trend; use BG meter for treatment decisions

## Cleaning and disinfecting

Clean your receiver when dirty or at least once a month.

### Clean

- Use one of these cleaners:**
  - Damp cloth with liquid hand soap and water
  - Bleach wipes
  - Ammonium wipes
- Wipe**
  - Using moderate pressure, wipe the receiver all over, 3 times up-and-down and 3 times side-to-side, to remove all dirt or soil.
- Let air dry**

### Disinfect

- Repeat cleaning instructions using a new bleach or ammonium wipe. If these wipes aren't available in your country, use an equivalent product.
- Wipe receiver until completely wet. Use wipe to keep receiver wet for 2 minutes.
- Let air dry.

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