Dexcom

Inserting Sensor

Instructions for Use

English

1 Choose Sensor Site by Age

(with advice from your healthcare provider)

- Age 7 years and older: Use back of upper arm or abdomen
- Age 24 months – 6 years: Use back of upper arm, abdomen, or upper buttocks

Avoid areas:

- With loose skin or without enough fat to avoid muscles and bones
- That get bumped, pushed, or you lie on while sleeping
- Within 8 centimeters of infusion or injection site
- Near waistband or with irritations, scarring, tattoos, or lots of hair

To keep sensor attached see dexcom.com/faqs

2 Clean and Dry

- Wash hands with soap and water. Dry.
- Rub site with an alcohol wipe.
- Wait until completely dry.

3 Unscrew Cap

- Don't touch inside applicator
- Don't use if damaged or previously opened

4 Use Applicator to Insert Sensor

- Relax any muscles near site
- Press and hold applicator firmly against skin and then push button

5 Remove Applicator

- Save applicator because you'll need its pairing code
- Screw cap back on applicator

6 Rub and Press

To help keep the patch on your skin:

- Rub firmly around patch 3 times
- Gently press on top of sensor for 10 seconds

7 Use Overpatch

(when needed)

The overpatch helps keep the sensor on your skin. Find the overpatch bundled with the instructions in the sensor box.

- Carefully pull off both clear liners, one at a time. Don't touch white adhesive area.
- Use colored tab to place overpatch around sensor.
- Rub around overpatch.
- Use tab to peel off colored liner.
- Rub around overpatch.
- Go back to the app or receiver to finish setup.

Adhesive Patch Care

- The longer you keep the patch dry and sweat-free in the first 12 hours, the longer it may stick to your skin
- When it gets wet, gently pat it dry as soon as you can
- If it peels off your skin, trim the peeled parts and put on a new overpatch or medical tape

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