

## CR1100 & CR1500

- Not Compatible with Other Models -

VERSION 03 – RELEASE DATE JAN 2022



codecorp.com

## Configuring CR1100 & CR1500 Barcode Readers

To configure a CR1100 or CR1500 to be assigned to a patient room or workstation-on-wheels, scan only one of the following barcodes:



CC004395\_02

Epic Patient Look up

&



CC005888\_01

Epic Alt Keypad Emulator

## CR1500 Feedback Settings

Scan the preferred feedback settings (not for a CR1100):



M20329\_01

Beep On  
Vibrate On



M20331\_01

Beep On  
Vibrate Off



M20330\_01

Beep Off  
Vibrate On

## CR100 and CR1500 Reader Settings



M20111\_01

Reset to factory defaults

## Firmware Version Information

Scan the following barcodes to identify firmware versions:



M20361\_01

Reader information